



**FOR INFORMATION USE ONLY**  
**Not to be used for performing the assay.**  
**Refer to the insert accompanying kit**

Catalogue # 150712 (en)

## **Saline Solution**

5 × 8 ml

For *in vitro* diagnostic use only

### **I. INTENDED USE**

The Saline Solution is intended to be used as a dilution and reconstitution buffer for coagulation assays.

### **II. REAGENT**

Each vial of Saline Solution contains 8 ml of buffered saline, with a preservative of 0.02% sodium azide.

### **III. ANALYSIS**

	<b>Specifications</b>
<b>pH determination:</b>	7.2-7.4

### **IV. STORAGE AND STABILITY**

Store at 2-8 °C. Use within 4 weeks from opening.

### **V. WARNINGS AND PRECAUTIONS**

The Saline Solution contains sodium azide, which may react with lead and copper plumbing to form highly explosive metal azides. Materials discarded into a sink should be flushed with a large volume of water to prevent azide build-up.

All wastes containing biological material should be properly labelled and stored separately from other wastes. Dispose of all waste materials according to prescribed international, national and local regulations.

The test should be used in conjunction with clinical observations and results of other laboratory tests

**KEY GUIDE TO SYMBOLS/ SIGNIFICATION DES SYMBOLES/ ERLÄUTERUNG DER SYMBOLE/ INTERPRETACIÓN DE LOS SÍMBOLOS/  
GUIDA AI SIMBOLI**



Use by  
A utiliser avant le  
Verw. bis:  
Utilizar antes de  
Usar entro.




Lot  
Lot



Ch.-B.:  
Lote  
Lotto.  
Catalogue number  
No. dans le catalogue  
Katalognummer  
No de catálogo  
No. di catalogo.



Manufacturer  
Fabricant  
Hersteller  
Fabricante  
Fabbicante

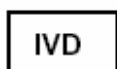

For in vitro diagnostic use  
Pour usage diagnostique in vitro  
in vitro diagnostikum  
Para uso diagnóstico in vitro  
Per uso diagnostico in vitro



Store at 2-8°C,  
Conserver à 2-8°C,  
Lagern bei 2-8°C,  
Conservar a 2-8°C,  
Conservare a 2-8°C.



Consult accompanying documents,  
Voir documents cijoins,  
Siehe beigefügte, Dokumente,  
Véanse los documentos djuntos,  
vedi documenti allegati.

**Manufactured by:**  
**Trinity Biotech plc,**  
**IDA Business Park,**  
**Bray,**  
**Co. Wicklow,**  
**Ireland.**  
**Tel: (353) 1 276 9800,**  
**Fax: (353) 1 276 9888,**  
**Web: [www.trinitybiotech.com](http://www.trinitybiotech.com)**

USA Enquiries:  
Trinity Biotech USA,  
1930 Innerbelt Business Center Drive,  
St. Louis,  
MO 63114,  
USA.  
Tel: (001) 800 325 3424